

H-B DURAC Liquid Crystal Disposable Forehead Stick-On Thermometer Strip

Catalog No. B62010-0000

Instructions for Use

How to Read

1. Remove backing liner and apply monitor to center of forehead, just above the eyebrows. Skin surface should be clean and dry.
2. Read monitor when the boxes stop changing color (generally 15 seconds), and while it is still on the forehead.
3. Green indicates the correct temperature; if green does not appear, the temperature is between blue and tan.

Intended Use

- Measure human forehead surface temperature and monitor temperature trends

Accuracy

- $\pm 1^{\circ}\text{F}$ (0.6°C)

Storage & Handling


- Store at 60 to 104°F in 15 to 95% RH
- Optimal storage at 60 to 80°F in 40 to 60% RH

Precautions

- Environmental temperatures might affect the performance of this product
- Product should be used indoors at normal room temperature
- Avoid exposure to UV light
- Discard after use

Liquid Crystal Forehead Temperature Trend Monitors are designed to be unbreakable noninvasive, quick reading devices that reliably indicate trends in core body temperature by measuring the surface temperature of the forehead. Liquid Crystal Forehead Temperature Trend Monitors indirectly measure core body temperature by measuring the forehead temperature, whereas clinical thermometers (oral/rectal) directly measure core body temperature. Unusual Liquid Crystal Forehead Temperature Trend Monitor readings should be verified by a clinical thermometer.

For external use only · Conforms to ASTM E-1061-01 · Not made with natural rubber latex

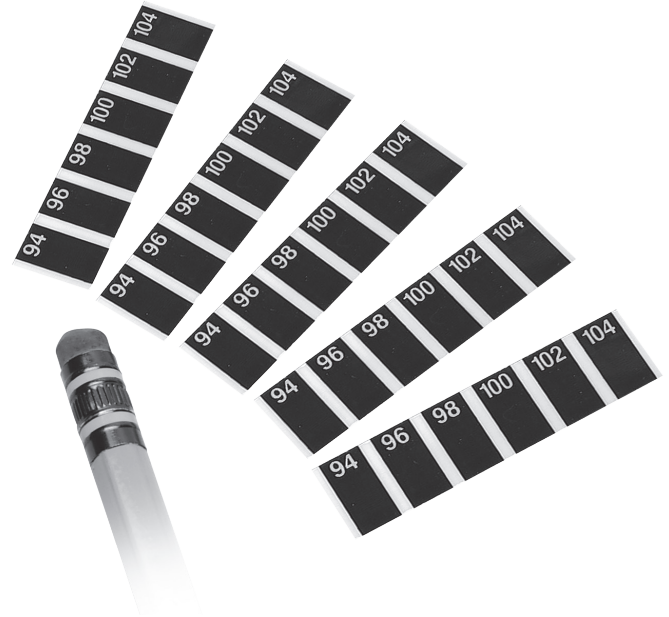

Your opinion counts! Write a product review at www.sp-belart.com



Bel-Art Products, a part of SP

661 Route 23 South, Wayne, NJ 07470 USA

Tel: 973-694-0500 | cservice.belart@spindustries.com

sp-belart.com

Individual use varies. Bel-Art Products/SP assumes no obligation or liability for any advice furnished by it or for results obtained with respect to these products. All advice is given and accepted at the user's risk. For warranty information visit belart.com/corporate.

